

# CAVITY CONDITIONER

## CAVITY CLEANING AGENT

Prior to use, carefully read  
the instructions for use.



For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

GC CAVITY CONDITIONER is a mild polyacrylic acid solution designed to remove the smear layer and to condition dentine and enamel, thus enhancing the bond between glass ionomer cement and tooth structure.

### CONTRAINDICATIONS

In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

1. After tooth preparation, apply GC CAVITY CONDITIONER to the bonding surfaces for 10 seconds using a cotton pellet or sponge.
2. Rinse thoroughly with water and dry. DO NOT DESICCATE.
3. Apply GC Fuji Bond LC, GC Fuji II LC (improved) or other glass ionomer as a bonding agent, base/liner or restorative material.
4. Close the bottle immediately after use.

### STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).  
(Shelf life : 3 years)

### PACKAGES

Bottle of 6g (5.7mL).

### CAUTION

1. In case of contact with oral tissue or skin, flush immediately with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.