

G-Premio BOND Technique Guide



Select etching mode



Self-etch



Selective etch

Etch enamel only for 10-15 sec., rinse for 5 sec. and dry.



Total-etch

Etch for 10-15 sec., rinse for 5 sec. and dry.

Bonding procedure



1. Apply G-Premio BOND and wait 10 sec.



2. Dry for 5 sec. (MAX air pressure).



3. Light-cure for 10 sec.



4. Start placing composite

G-Premio BOND Technique Guide



Intraoral repair



1. Roughen surface. Rinse and dry.




2. For all glass ceramics and hybrid ceramics, apply G-Multi Primer. Dry.



3. Apply G-Premio BOND to all surfaces and wait 10 sec.



4. Dry for 5 sec. (MAX air pressure).

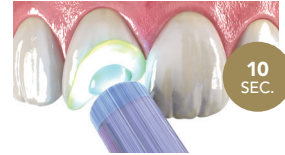

5. Light-cure for 10 sec. Begin composite application to complete repair.

Hypersensitivity treatment



1. Apply G-Premio BOND. Wait 10 sec.



2. Dry for 5 sec. (MAX air pressure).



3. Light-cure for 10 sec.



4. Remove unpolymerised layer using a cotton pellet.