

ARTICULATING PAPER

BLUE

RED

12 SHEETS×10

Prior to use, carefully read the directions for use.

CAUTION

1. Articulating Paper is for single use only. To prevent cross contamination between patients, do not reuse the Articulating Paper.
2. For use only by a dental professional in the recommended indications.
3. This product should be used only for the applications described in the instructions.

TYPE

Size:

22mm x 120mm

Color:

RED

BLUE

Contents:

12 sheets in 1 booklet x 10

INDICATIONS

Marking the occlusal condition of natural teeth, artificial teeth and other restoratives.

DIRECTION FOR USE

1. Dry out occlusal surface of tooth structure (artificial teeth and other restoratives).
2. To adjust occlusal contact by grinding on higher contact area, bite several times at centric occlusion and grind color marked area.
3. To diagnosis difference of occlusal contact on each mandibular position parallel, occlusal contact for each mandibular position may be easily recognized with marking by different colored product.