Prior to use, carefully read the instructions for use.

NEW METAL STRIPS

INTERPROXIMAL FINISHING STRIPS

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- For finishing the interproximal surfaces of fillings.
- Tooth separation.
- Preparation of abutment teeth.

DIRECTIONS FOR USE

- Select a suitable grade.
- Insert between the teeth.
- Pull back and forth using light pressure only.

Notes

- NEW METAL STRIPS have sharp edges. Wear gloves to protect hands and take care not to injure the patient's oral tissues, face, etc.
- May be re-used. Clean in an ultrasonic cleaner or burn out in a flame and sterilize by autoclaving or chemical means.

TYPES

- # 200 mesh (red); # 300 mesh (blue); # 600 mesh (green); # 1000 mesh (yellow)
- Width - A (2.6 mm); B (3.3 mm); C (4.0 mm) : Length - 133 mm
- Thickness - # 200 (140 µm); # 300 (90 µm); # 600 (70 µm); # 1000 (50 µm)

PACKAGES

- All same mesh - Box of 12 strips (4 pieces each of 3 widths, A, B and C)
- All same mesh - Box of 12 strips (12 pieces - all one width)
- Assortment - Box of 12 strips (1 piece each width and each mesh)

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en
 as well as to our internal vigilance system: vigilance@gc.dental
 In this way you will contribute to improve the safety of this product.

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