Prior to use, carefully read the instructions for use.

UNIFAST™ Trad

GENERAL PURPOSE ACRYLIC RESIN

GC UNIFAST Trad is a self curing, fast setting acrylic resin for use in a variety of dental applications. For use in either pressure or paint-on techniques.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Repair of dentures.
2. Construction of temporary crowns.
3. Impression trays for individual teeth.
4. Orthodontic plates, splints, check bite and other general purposes.

CONTRAINDICATIONS




Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

A. PRESSURE TECHNIQUE

The recommended powder / liquid ratio is 1 g of powder (1st graduation of powder measure) to 0.5 mL (1st graduation of liquid measure) of liquid.

1. Dispense the required amount of liquid into the rubber cup as follows:
 - a) Remove the protective cap and screw on the dispensing nozzle (Fig. 1)
 - b) Pour into the measure provided (Fig. 2) until the bottom of the meniscus is level with the required graduation (Fig. 3).
 - c) Replace the protective cap to prevent evaporation.
2. Add the powder and mix thoroughly for 10-15 seconds.
3. The mixture reaches a dough-state 20-30 seconds after mixing. All manipulation should be finished before 2 minutes after mixing when setting starts.

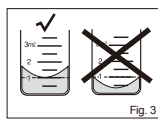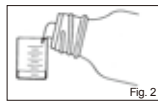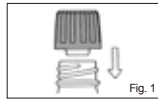B. PAINT-ON TECHNIQUE

Use standard techniques.

NOTES

1. Do not use GC UNIFAST Trad powder or liquid with components of other brands of acrylic resin.
2. Remove residual resin with a dry gauze after each application.
3. Mop up liquid spills with a dry cloth.

STORAGE

Recommended for optimal performance, store in a cool place (2-37°C / 36-98°F) away from direct sunlight.

PACKAGES

1. Bottles of powder, 35 g, 100 g, 250 g, 1 kg
2. Bottles of liquid, 100 g (104 mL), 250 g (260 mL)

SHADES

1. Crown colours : Ivory, Deep Ivory, Clear ivory.
2. Gingival colours : Pink # 3, Dark pink # 4, Natural light pink # 6, Natural dark pink # 7, Live pink # 8 (veined), Pink X

CAUTION

1. The liquid is HIGHLY FLAMMABLE. Avoid sources of ignition.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://sea.gcasiadental.com>
They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSER. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

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GC ASIA DENTAL PTE. LTD.
GC KOREA CO., LTD.
GC AUSTRALASIA DENTAL PTY. LTD.