COE-SOFT™

RESILIENT DENTURE LINER

For use only by a dentist or physician in the recommended indications.

RECOMMENDED INDICATIONS

A temporary lining for acrylic dentures. For use in chairside procedures.

CONTRAINDICATIONS

Do not use on patients with known sensitivity to methacrylate. In case of allergy, consult a physician.

DIRECTIONS FOR USE

Product Handling:

Use caution when opening. COE-SOFT Liquid is a highly flammable liquid and vapor; keep away from heat, hot surfaces, sparks, flames and other ignition sources. Use with adequate ventilation. Do not ingest or breathe vapor. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Check bottle with liquid before using. Make sure that upon removal of cap, the bottle opening has no obstructions. Some bottles may contain an insert (as a seal). The insert must be in the cap and not obstructing opening.


In case of contact, flush eyes with water for at least 15 minutes. Get prompt medical attention. If overcome by vapor, move to fresh air. If breathing is difficult, administer oxygen and seek medical attention. If swallowed, dilute by giving water to drink and seek medical attention. In case of contact with skin wash thoroughly with soap and water. Store securely. Store in a well-ventilated place. Keep cool. Dispose of all waste according to local regulations.

I. Preparation of the denture:

Relieve and roughen the area of the denture to be relined. Clean and dry the denture thoroughly. Coat labial and buccal surfaces of the denture with COE LUBRICANT. Do not apply coating within 3mm (1/8 inch) of the peripheral border. If the denture has plastic teeth also protect them with COE LUBRICANT. Note: COE-SOFT will not adhere to surfaces coated with COE LUBRICANT.

II. Preparation of COE-SOFT:

Recommended powder / liquid ratio is 11g powder to 8ml liquid. Pour the liquid into the large mixing cup. Then add the powder slowly. Stir mixture thoroughly for 30 seconds. A suitable spatula is provided for this purpose. To avoid introducing bubbles into the mixture, do not spatulate for more than 30 seconds. Do not whip.

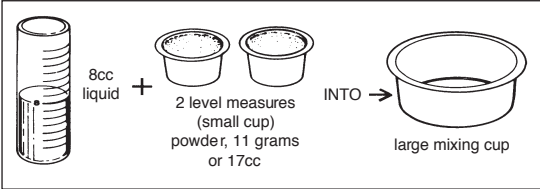III. Application:

Spread the mixture of COE-SOFT over the area to be relined. Seat the denture in the manner of taking an impression and instruct the patient to close lightly into occlusion. After 3 minutes instruct the patient to move lips and cheeks so that a muscle trimmed periphery is obtained. Remove the denture and rinse under cold water. Trim away excess material. Re-seat the denture and instruct the patient to close FIRMLY into occlusion and to hold this position for 5 minutes. Remove the denture and rinse again in cold water.

IV. Finishing:

When curing is complete, trim away excess. For smoothing the edges use a hot spatula.

V. Patient Advice:

Advise the patient NOT to use a brush or abrasive (such as toothpaste) on the lining. Cleaning is best achieved by holding it under cold running water and wiping with wet cotton.

STORAGE

Recommended for optimal performance; store in a dry location at room temperature (70° to 77°F; 21° to 25°C).

PACKAGES

COE-SOFT Professional Package

COE-SOFT Powder, 170g

COE-SOFT Liquid, 6fl oz (177mL)

COE-SOFT Economy Package

COE-SOFT Powder, 5lbs (2.27kg)

COE-SOFT Liquid, 32fl oz (946mL)

Note: Not all packaging types are available in all countries.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas:

<http://www.gcamerica.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients the bottles and measuring devices require mid-level disinfection. Immediately after use inspect the bottles, measuring devices and label for deterioration. Discard if damaged.

DO NOT IMMERSE. Thoroughly clean bottles and measuring devices to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects - Reporting

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

as well as to our internal vigilance system:

vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

Last Revised: 09/2019

R_XOnly

GC

MANUFACTURED BY
GC AMERICA INC.
3727 West 127th Street, Aspi, IL 60803 U.S.A.
TEL: +1 708-587-9200
www.gcamerica.com

DISTRIBUTED BY

GC EUROPE N.V.
Researchpark Hasselt-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32-16-74-10-00

GC SOUTH AMERICA
Rua Heliodora, 399 - Sarreana - São Paulo, Sp - Brasil
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.899/0001-61

RES.P. TEC. Mayara De Santos Ribeiro - CRO/SOP: 105.982

PRINTED IN THE U.S.A.

N 940919

Prior to use, carefully read the instructions for use.

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