Prior to use, carefully read the instructions for use.

EN

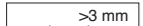
Fig. 1B1d. 1

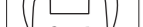
Fig. 2B1a. 2

Fig. 2B1a. 3

Fig. 2B1a. 4

Fig. 2B1a. 5

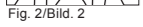
Fig. 2B1a. 6

Fig. 2B1a. 7

Fig. 2B1a. 8

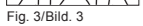
Fig. 2B1a. 9

Fig. 2B1a. 10

Fig. 2B1a. 11

Fig. 2B1a. 12

Fig. 2B1a. 13

Fig. 2B1a. 14

Fig. 2B1a. 15

Fig. 2B1a. 16

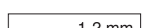
Fig. 2B1a. 17

Fig. 2B1a. 18

Fig. 2B1a. 19

Fig. 2B1a. 20

Fig. 2B1a. 21

Fig. 2B1a. 22

Fig. 2B1a. 23

Fig. 2B1a. 24

Fig. 2B1a. 25

Fig. 2B1a. 26

Fig. 2B1a. 27

Fig. 2B1a. 28

Fig. 2B1a. 29

Fig. 2B1a. 30

everX Posterior™

FIBER REINFORCED COMPOSITE FOR DENTIN REPLACEMENT

Use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS
 everX Posterior is suitable for use as the reinforcing material for direct composite restorations, especially in large posterior cavities, for example:
 1. Cavities including 3 surfaces or more
 2. Cavities with missing cusps
 3. Deep cavities (including class I, II and endodontically treated teeth)
 4. Cavities after amalgam replacement
 5. Cavities where onlays & inlays would also be indicated
 Note:
 everX Posterior should always be covered with a layer of light-cured universal restorative composite, for sufficient wear resistance

CONTRAINDICATIONS

- Direct pulp capping
- Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymers
- At least one horizontal dimension of the cavity should exceed 3 mm.
- The product is not suitable for patients with a history of hypersensitivity to methacrylate monomers.
- Do not use as final approximal and surface composite layer.

DIRECTIONS FOR USE

- Shade Selection
Clean the tooth with pumice and water. Shade selection of the final composite enamel layer should be made prior to isolation.
- Cavity Preparation (Fig. 1)
Prepare cavity using standard techniques. Dry by gently blowing with oil free air. The use of rubber dam is recommended to isolate prepared tooth from contamination with saliva, blood or sulcus fluid.

Preparation of everX Posterior (Fig. 3) – see note for Class II and large cavities

Pulp capping, use calcium hydroxide.

Bonding treatment (Fig. 2)
 For bonding everX Posterior to enamel and/or dentin, use a light-cured bonding system such as G-Premo BOND, G-g-nial Bond or G-BOND. Follow manufacturer's instructions.

Placement of everX Posterior (Fig. 3) – see note for Class II and large cavities

Insert everX Posterior into the Unitip APPLIER II or equivalent.
 Remove the cap, and extrude everX Posterior material directly into the cavity.

Remove the cap, and extrude everX Posterior material directly into the cavity.

Light curing everX Posterior in the cavity, using suitable instruments, until it has been adapted tightly against the cavity floor and walls. To prevent premature curing, avoid bright light in the work area. When placing everX Posterior material in the cavity, ensure that there is enough space for the overlying composite on all surfaces of the restoration (refer to point 6).

Use a suitable matrix system to establish the proximal contact and note – Class II and large cavities (Fig. 3a, 3b)

When placing everX Posterior in Class II or larger cavity, build first the missing walls by using light-curing restorative composite before placing everX Posterior. The wall should be thick enough to withstand the application pressure of everX Posterior.

Material may be hard to extrude immediately after removing from cold storage. Prior to use, leave to stand for a few minutes at normal room temperature.

Light curing everX Posterior using a light curing unit. Keep light guide as close as possible to the surface. Refer to the following chart for irradiation Time and Effective Depth of Cure.

Curing device	Irradiation Time	Depth of cure
High Power LED Light (>1200 mW/cm²)	10 sec.	4 mm
Halogen/ Normal LED Light (700 mW/cm²)	20 sec.	4 mm

Note:
 Lower light intensity may cause insufficient curing or discoloration of the material.

Placement of the overlying composite (Fig. 5)
 Fill the remaining cavity space with a wear- resistant and polishable restorative composite, such as G-g-nial Posterior or GC KALORE. The composite layer should be of 1-2 mm on the occlusal surface. Light-cure as instructed by the manufacturer.

Finishing and Polishing
 Check and adjust the occlusion. Finish and polish using standard techniques.

SHADE
 everX Posterior is available in one universal shade.

STORAGE
 Recommended for optimal performance, store in a cool and dark place (4-25°C / 39.2-77.0°F) away from high temperatures or direct sunlight.

PACKAGES
 50 units (0,25 g / 0,13 mL)
 50 units (0,25 g / 0,13 mL)

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. Take care to avoid ingestion of the material.

4. Prior to starting the filling process, partially extrude paste outside of the patient's mouth and away from the patient to ensure paste is flowing properly.

5. Do not mix with other similar products.

6. For infection control reasons, Unitips are for single use only.

7. Wear rubber-fleece plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.

8. Wear protective eye glasses during light curing.

9. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.

10. Avoid getting material on clothing.

11. In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light curing.

12. Do not use everX Posterior in combination with eugenol containing materials as eugenol may hinder everX Posterior from setting.

13. Do not wipe the unitip with excessive alcohol as this may cause contamination of the paste.

14. Dispose of all wastes according to local regulations.

15. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

16. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
<http://www.gceurope.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING:
 MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects: Any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instructions for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en
 as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

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