Prior to use, carefully read
the instructions for use.

EN

GC Fuji™ IX GP FAST

RADIOPAQUE POSTERIOR GLASS IONOMER RESTORATIVE CEMENT IN CAPSULES

GC Fuji IX GP FAST is a fast-setting version of GC Fuji IX GP CAPSULE.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Class I and II restorations in deciduous teeth.
2. Non-invasive Class I and Class II restorations in permanent teeth.
3. Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
4. Class V and root surface restorations.
5. Core build-up.


CONTRAINDICATIONS

1. Pulp capping.
2. Avoid use of this product in patients with known allergies to glass ionomer cement.


DIRECTIONS FOR USE	Fast	Original
Powder / Liquid Ratio (g/g)	0.40 / 0.11	0.40 / 0.11
Mixing Time (sec.)	10"	10"
Time of initial extrusion after mixing	10 seconds maximum	
Working Time (23°C/73°F) (min., sec.)	1'15"	2'00"
Net Setting Time (37°C/99°F) (min., sec.)	2'00"	2'20"
Finishing Time (37°C/99°F, from start of mixing) (min., sec.)	3'00"	6'00"

1. CAVITY PREPARATION

- a) Prepare with standard techniques. Extensive mechanical retention is unnecessary. For pulp capping use calcium hydroxide.
- b) Apply CAVITY CONDITIONER (10 seconds) or DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge (Fig. 1).
- c) Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).

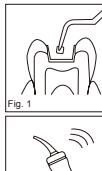**2. CAPSULE ACTIVATION AND MIXING**

- a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 2).
- b) To activate the capsule, push the plunger until it is fully recessed with the main body and hold it down for 2 seconds (Fig. 3).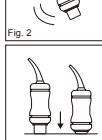

Note:


1) Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.

2) The capsule should be activated just before mixing and used immediately.

c) Immediately set it into a mixer (or an amalgamator) and mix for 10 seconds (+/- 4,000 RPM) (Fig. 4).

**3. RESTORATIVE TECHNIQUE**

- a) Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
- b) Make two clicks to prime the capsule then syringe (Fig. 5). The working time is 1 minute 15 seconds from start of mixing at 23°C (73°F). Higher temperatures will shorten working time.

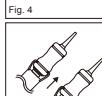- c) Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation (Fig. 6).

d) Form the preliminary contour, and cover with a matrix if required.

e) During the first 3 minutes from start of mix extra care should be taken to avoid moisture contamination or drying-out. In case this cannot be guaranteed, immediately apply or GC Fuji Varnish (blow dry) or GC Fuji Coat LC (litchi cure).

Note:

1) To adjust the direction of the nozzle, hold the applier with the capsule towards you and turn the capsule body.

2) To remove the used capsule, push the applier release button. Twist the capsule and pull upwards.

4. FINISHING

- a) Final finishing under water spray using standard techniques can begin approximately 3 minutes after start of mixing (Fig. 7).
- b) Apply a final coat of G-COAT PLUS and light cure for 20 seconds (Fig. 8). Alternatively, apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure).

Refer to the respective instructions for use.

c) Instruct the patient not to apply pressure for 1 hour.

5. STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C) (39.2-77.0°F).

SHADES

- A1, A2, A3, A3.5, B2, B3, C4

Shade numbers according to Vita® shade guide.

Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

1. Single shade package: 50 capsules

2. Assortment package: 50 capsules

(10 each of A1, A2, A3, A3.5, B2, B3)

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge soaked in alcohol. Flush with water.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gc-europe.com>

They can also be obtained from your supplier.

Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

vigilance@gc.dental

MANUFACTURED BY
GC CORPORATION
7-1 Hasuruma-cho, Itabashi-ku, Tokyo 174-8585, JapanEU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 3240, Interleuvenlaan 33,
B-3000 Leuven, Belgium TEL: +32 16 74 10 00DISTRIBUTED BY
GC CORPORATION
7-1 Hasuruma-cho, Itabashi-ku, Tokyo 174-8585, JapanGC ASIA DENTAL PTE. LTD.
5 Tamplines Central 1, #06-01 Tamplines Plaza, Singapore 529541
TEL: +65 6546 5788GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Bankstown, NSW 2219, Australia
TEL: +61 2 9301 8200

PRINTED IN JAPAN

Prior to use, carefully read
the instructions for use.

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10"

10"

Time of initial extrusion after mixing

10 seconds maximum

Working Time (23°C/73°F) (min., sec.)

1'15"

2'00"

Net Setting Time (37°C/99°F) (min., sec.)




2'00"

Finishing Time (37°C/99°F, from start of mixing) (min., sec.)

3'00"

6'00"

of mixing)

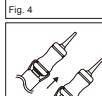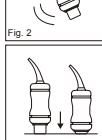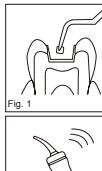d) After activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 1).

e) To activate the capsule, push the plunger until it is fully recessed with the main body and hold it down for 2 seconds (Fig. 2).

f) Immediately set it into a mixer (or an amalgamator) and mix for 10 seconds (+/- 4,000 RPM) (Fig. 3).

g) Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation (Fig. 4).

h) Form the preliminary contour, and cover with a matrix if required.

i) During the first 3 minutes from start of mix extra care should be taken to avoid moisture contamination or drying-out. In case this cannot be guaranteed, immediately apply or GC Fuji Varnish (blow dry) or GC Fuji Coat LC (litchi cure).

j) To adjust the direction of the nozzle, hold the applier with the capsule towards you and turn the capsule body.

k) To remove the used capsule, push the applier release button. Twist the capsule and pull upwards.

l) Final finishing under water spray using standard techniques can begin approximately 3 minutes after start of mixing (Fig. 5).

m) Apply a final coat of G-COAT PLUS and light cure for 20 seconds (Fig. 6). Alternatively, apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure).

n) Refer to the respective instructions for use.

o) Instruct the patient not to apply pressure for 1 hour.

5. STORAGEN

Recommended for optimal performance, store in a cool and dark place (4-25°C) (39.2-77.0°F).

SHADES

- A1, A2, A3, A3.5, B2, B3, C4

Shade numbers according to Vita® shade guide.

Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

1. Single shade package: 50 capsules

2. Assortment package: 50 capsules

(10 each of A1, A2, A3, A3.5, B2, B3)

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge soaked in alcohol. Flush with water.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

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