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Prior to use, carefully read the instructions for use. **EN**

GC TISSUE CONDITIONER

TISSUE CONDITIONING AND RELINING MATERIAL

For use by or under the supervision of a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Tissue conditioning.
- Temporary relining.
- Functional impression taking.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer, ethanol or aliphatic acid ester-based plasticizer.

DIRECTIONS FOR USE

- PREPARATION
 - Check the patient's oral conditions and the denture in use. Adjust misfitting areas with a carbide bur, clean and dry. Create sufficient space so GC TISSUE CONDITIONER can be applied at a minimum thickness of 1 mm in order to maintain the elasticity.
 - Apply COCOA BUTTER or petroleum jelly to the areas where the denture should not be relined.
- POWDER / LIQUID DISPENSING
 - The standard powder / liquid ratio is 2.4 g / 2 mL (1 unit of powder to 2 units of liquid). Dispense the required amounts of powder and liquid into the rubber cup.

Nota:

 - a) Using less liquid will result in a shorter gel phase and a more viscous mixture. More liquid will result in a longer gel phase and a less viscous mixture.
 - b) Powder may form aggregates in the bottle under certain storage conditions. This will not affect the function.
 - c) Close the bottles of powder, liquid and coating agent tightly immediately after use.
- MIXING
 - Mix powder and liquid for 30-60 seconds until the mixture reaches a creamy consistency.
- APPLICATION
 - 1) The coating agent can be used as a bonding agent to improve the bond strength and to avoid possible delamination. Apply the coating agent with the brush only in the areas where a stronger bond is needed. In this case, GC TISSUE CONDITIONER cannot be peeled off by hand.
 - 2) Apply the mixture evenly to the fitting surfaces of the prepared denture.

Nota:

 - a) To avoid sticking of the material, lubricate fingertips or instruments with water. Do not use lotion or petroleum jelly.
 - b) Prevent excess from flowing into the throat.
 - 3) Seat the denture in the mouth. Ask the patient to gently occlude in the centric occlusion and move the lips and mandibles to muscle trim. After keeping the denture for about 5 minutes in this position, remove and inspect for excess and deficiencies.
 - 3) Remove any excess using a sharp instrument.
- FINISHING
 - 1) Wash away saliva from the relined denture under running water.
 - 2) Apply the coating agent to the relined surfaces of the denture. Dry the coating agent thoroughly with an air syringe or leave it undisturbed for 4-5 minutes.

Nota:

 - To improve the marginal seal, coating agent can also be applied to the GC TISSUE CONDITIONER / denture interface.
 - 3) Reseat the denture in the mouth.
- IMMERSION
 - a) Immersion of the denture in water before seating in the mouth will reduce unwanted odor.
 - b) Instruct the patient to clean the relined areas with fingers under running water. The use of a brush may damage the relined areas.
 - Care should be taken not to use denture cleaning agents for a prolonged period of time.

- REMOVAL
 - It is recommended to remove the material or replace it with a new one after a few days or weeks depending on each clinical case.
 - GC TISSUE CONDITIONER can be peeled off by hand. Use a sharp instrument or bur to remove the area where the coating agent was used as a bonding agent.
- STORAGE
 - Recommended for optimal performance, store in a cool, dark place (4-25°C) (39.2-77.0°F) away from direct sunlight and high temperature.
- COLORS
 - Live pink, White
- PACKAGES
 - 1. 1-1 Package
 - Kit contains: Powder 90 g (1), Liquid 90 g (101 mL) (1), Coating agent 12 g (15 mL) (1), Powder measure (1), Plastic liquid syringe No.1 (1), Brush No.7 (1), Plastic spatula (1), Rubber cup (Large) (1)
 - 2. Refill Packages
 - a. GC TISSUE CONDITIONER powder 90 g (in each shade)
 - b. GC TISSUE CONDITIONER liqui 90 g (101 mL)
 - c. GC TISSUE CONDITIONER coating agent 12 g (15 mL)

CAUTION

- The liquid is flammable. Avoid sources of ignition. Do not keep or use near fire or high temperature.
- Use in a well ventilated place. Replace cap immediately.
- Avoid contact with eyes. In case of contact, immediately flush with water and seek medical attention.
- Avoid direct contact of liquid, mixture or coating agent on skin. In case of contact, wipe with a cotton pellet or sponge soaked in alcohol and flush with water.
- Coating agent may turn yellow with time, but this will not affect its function.
- Do not mix with other products.
- In rare cases the product may cause sensitivity in some people.
- If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

http://www.gceurope.com
They can also be obtained from your supplier.

CLEANING AND DISINFECTING:
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERGE. Thoroughly clean water to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

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