

# Miracle Mix<sup>®</sup> CAPSULE



Silver Alloy-glass Ionomer Cement Restorative in Capsules

**Change for the better!**  
**Available in convenient  
delivery system!**





# Miracle Mix® CAPSULE

Silver Alloy-glass Ionomer Cement for Core Build-up in Capsules

## Greater Ease and Efficiency

GC Miracle Mix CAPSULE, a capsulated silver alloy-glass ionomer cement, is designed to simplify core building. The capsule contains pre-measured silver alloy-glass ionomer cement in powder/liquid form. The GC Miracle Mix CAPSULE, used in conjunction with The GC CAPSULE APPLIER II and an ordinary amalgamator, allows procedures to be performed with greater ease and efficiency, from dispensing and mixing to core building.



### Features

- The capsule system simplifies procedures, from dispensing and mixing to core building.
- Strong chemical bonding maximizes marginal sealing.
- Excellent fluoride release\*.
- Outstanding radiopacity facilitates post-diagnosis.
- High strength and durability.


\* As a glass ionomer, Miracle Mix CAPSULE releases fluoride into the tooth structure, fluoride uptake has been shown to minimize the incidence of dental caries.\*\*

\*\* *Restorative Materials Containing Fluoride*, "Council on Dental Materials, Instruments and Equipment, JADA, Vol. 116, May 1988.


### Applications

- Core build-ups
- Restoration of primary teeth
- Cases requiring radiopacity
- Bases under composite resin and amalgam in molars
- Temporary fillings, and others

### Directions for use

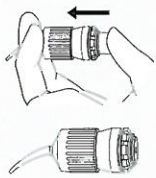

1. Shake the capsule to loosen powder.  
To activate the capsule, push the plunger until it is flush with the main body.

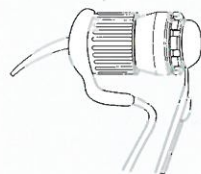

2. Promptly set the capsule on the capsule mixer. Mix for 10 seconds.



3. Mount the mixed capsule on the GC CAPSULE APPLIER II. Extrude the mixture by pulling the lever 4-5 times, then apply.



4. The cement will set in 5 minutes. Perform shape corrections using a diamond point with water.



5. The core impression can now be taken. For temporary cementation, GC Freegenol Temporary Pack is recommended. In addition, GC Fuji I or Fuji I CAPSULE is recommended for final cementation.

### Options:

## CAPSULE APPLIER II

The GC CAPSULE APPLIER II is most suitable for use with the GC Miracle Mix CAPSULE to apply material easily and efficiently.

## CAPSULE MIXER CM-I

For best mixing results, the GC CAPSULE MIXER CM-I is recommended for mixing the GC Miracle Mix CAPSULE.



CAPSULE MIXER CM-I

### Package

Miracle Mix CAPSULE: 50 capsules\*

\*Average contents per capsule: 0.52g silver alloy-glass ionomer cement powder and 0.11g (0.09ml) liquid

### GC CORPORATION

76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174, Japan Phone: 03-3558-5181 Fax: 03-3966-1470  
<http://www.gcdental.co.jp/>

For further information, please contact

GC BELGIUM N.V. Research-Park, 13 Inlerleuvenlaan, B-3001 Leuven, Belgium Phone: 016-402700 Fax: 016-400214

GC GERMANY GmbH Robert-Bosch-Straße 17a 63477 Mainlat-Dörnigheim, Germany Phone: 06181-947600 Fax: 06181-947629

GC FRANCE 8, Avenue du Bouton d'Or, F-94380 Bonneuil S/Marne, France Phone: 01-4980-3791 Fax: 01-4980-3790

GC ITALY S.r.l. Via Calabria, 1 (Sesto Uteriano) I-20098 S. Giuliano Milanese (MI), Italy Phone: 02-98282068 Fax: 02-98282100

GC AMERICA INC. 3737 West 127th Street, Alsip, IL #0803 U.S.A. Phone: 708-597-0900 Fax: 708-371-5103

GC TRADING CORP. Peeres Bldg. #222, Chungjeongno 3-Ga, Seodaemun-Gu, Seoul, Korea Phone: 02-313-2272 Fax: 02-313-2275

GC TAIWAN CORP. 17 Lane 327, Chung-Shan RD, Ser 2, Chung-Ho City, Taipei Hsien, Taiwan Phone: 2-246-3315 Fax: 2-245-5072

GC ASIA DENTAL PTE. LTD. 19 Lojong Way, #06-27 Singapore 508724 Phone: 65-5467563 Fax: 65-5467577