# GC Fuji™ BULK

RADIOPAQUE POSTERIOR GLASS IONOMER RESTORATIVE CEMENT IN CAPSULES

For use only by a dental professional in the recommended indications.

## RECOMMENDED INDICATIONS

1. Class I and II restorations in deciduous teeth.
2. Non-load bearing Class I and Class II restorations in permanent teeth.
3. Base material for load bearing situations in Class I and Class II cavities using a sandwich laminate technique.
4. Transitional and intermediate restorations.
5. Class V and root surface restorations.
6. Core build-up.

## CONTRAINDICATIONS

1. Pulp capping.
2. Avoid use of this product in patients with known allergies to glass ionomer cement.

## DIRECTIONS FOR USE

|  |                    |
|--|--------------------|
| Powder / Liquid Ratio (g/g)                                    | 0.4 / 0.11         |
| Mixing Time (sec.)   | 10"                |
| Time of initial extrusion after mixing                         | 10 seconds maximum |
| Working Time (23°C /73°F) (min., sec.)                         | 1'15"              |
| Net Setting Time (37°C /99°F) (min., sec.)                     | 2'00"              |
| Finishing Time (37°C /99°F, from start of mixing) (min., sec.) | 2'00"              |

## 1. CAVITY PREPARATION

- a) Prepare tooth using standard techniques. Extensive mechanical retention is unnecessary. For pulp capping use calcium hydroxide.
- b) Apply CAVITY CONDITIONER (10 seconds) or DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge.
- c) Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).

## 2. CAPSULE ACTIVATION AND MIXING

- a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder.
- b) To activate the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds.  
Note:
  - 1) Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.
  - 2) The capsule should be activated just before mixing and used immediately.
- c) Immediately set it into a mixer (or an amalgamator) and mix for 10 seconds (+/- 4,000 RPM).

## 3. RESTORATIVE TECHNIQUE

- a) Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
- b) Make two clicks to prime the capsule then syringe. The working time is 1 minute 15 seconds from start of mixing at 23°C (73°F). Higher temperatures will shorten working time.
- c) Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation.
- d) Form the preliminary contour, and cover with a matrix if required.
- e) During the first 2 minutes from start of mix extra care should be taken to avoid moisture contamination or drying-out. In case this cannot be guaranteed, immediately apply G-COAT PLUS (light cure for 20 sec.) or EQUIA Forte Coat (light cure for 20 sec.) or GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure).  
Note:
  - 1) To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
  - 2) To remove the used capsule, push the applicator release button. Twist the capsule and pull upwards.

## 4. FINISHING

- a) Final finishing under water spray using standard techniques can begin approximately 2 minutes after start of mixing.
- b) Apply a final coat of G-COAT PLUS (light cure for 20 sec.) or EQUIA Forte Coat (light cure for 20 sec.) or GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.).
- c) Instruct the patient not to apply pressure for 1 hour.

## STORAGE

Recommended for optimal performance, store GC Fuji BULK (in the original aluminium foil) in a cool and dark place (4-25°C) (39.2-77.0°F).

## SHADES

Universal

## PACKAGES

Single shade package: 50 capsules  
Average contents per capsule: 0.4g powder and 0.11g (0.09mL) liquid  
Minimum net volume of mixed cement per capsule: 0.14mL  
Option: GC CAPSULE APPLIER (1 piece)

## CAUTION

1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
3. In case of contact with eyes, flush immediately with water and seek medical attention.
4. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
5. If swallowed: rinse mouth. Do not induce vomiting.
6. Dispose of all waste according to local regulations.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gcasiadental.com>

They can also be obtained from your supplier.

Last revised: 06/2019

MANUFACTURED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

GC ASIA DENTAL PTE. LTD.  
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541  
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.  
1753 Botany Rd, Banksmeadow, NSW 2019, Australia  
TEL: +61 2 9301 8200

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