Prior to use, carefully read the instructions for use. [EN]

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everX Flow™

KORTE FIBER VERSTERKT FLOWABLE COMPOSITE FOR DENTIN REPLACEMENT

Use only by a dental professional in the recommended indications.

INDICATIONS FOR USE
everX Flow is a reinforcing dentin replacement material suitable for:
1. All direct composite restorations including large posterior cavities, deep cavities and end-treated teeth, cavities with missing cusps or after amalgam removal and cavities where inlays and onlays would also be recommended.
2. Core build-up.
Note:
Although everX Flow restores a high strength and fracture toughness, it should always be fully covered with a layer of light-cured universal restorative composite to ensure sufficient wear resistance.

CONTRAINDICATIONS
1. Direct pulp capping.
2. Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.
3. Do not use as final approval and surface composite layer. Always cover with a layer of light-cured universal composite.

DIRECTIONS FOR USE
1. Preparation
a) Hold the syringe upright and remove the cap by turning counterclockwise.
b) Promptly and securely attach the dispensing tip to the syringe by turning clockwise.
c) Place the light protective cap until use.
Note:
Take care not to attach the dispensing tip too tightly. This may damage its screw. In order to ensure a tight connection, make sure that the threads are free of remnant material.
2. Shade Selection
Clean the tooth with pumice and water. Select the appropriate shade of the restoration and use a color guide.
Light-curing for the respective depth of cure of each shade.

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