

GC Capsule Activation Instructions



Shake or tap its side to loosen powder.



Make sure to depress plunger and hold it down for **2 seconds**.



Mix for **10 seconds**.



Two clicks to prime capsule then syringe slowly.

NEW

PUSH AND MIX



The GC Capsule Range



GC



Since 1921
Towards Century of Health

GC Australasia Dental Pty Ltd
1753 Botany Road Banksmeadow
NSW 2019 Australia

T: +61 2 9301 8200
E: info.australasia@gc.dental
www.gcaustralasia.com