Prior to use, carefully read the instructions for use.

EN

## ISO FUNCTIONAL

REBASING AND RELINING IMPRESSION MATERIAL

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

Because of its unique slow setting characteristics ISO FUNCTIONAL enables production of extremely accurate impressions. This makes it particularly useful as an impression material for rebasing and muscle trimming. May also be used for general impressions such as dentures, root surfaces and posts.

### DIRECTIONS FOR USE REBASING

- Remove the flange and tissue contact surface from the denture to a depth of 2-3 mm using steel burs or abrasive points.
- Cover the relieved surface of the denture with ISO FUNCTIONAL which has been softened over a flame.
- Soften again by passing through hot water at 45°C (113°F). Insert the denture into the mouth and instruct the patient to bite and hold the denture in centric occlusion. Perform muscle trimming movements.
- The denture should be retained in the mouth for 1 day to allow a truly functional impression to be formed under normal chewing conditions.
 

Note :

Advise patients to avoid hot drinks during this period.
- The impression formed is used in standard rebasing procedures.

#### NOTES

- ISO FUNCTIONAL has a fast colour, no taste and no smell and is non-irritating to the patient's mouth.
- To avoid ISO FUNCTIONAL becoming too sticky do not overheat.
- If ISO FUNCTIONAL adheres to the denture, let it set hard and remove with laboratory burs and hand instruments. Remove residual traces using solvent.

#### STORAGE

Recommended for optimal performance, store at temperature of (4-25°C) (39.2-77.0°F).

#### PACKAGES

ISO FUNCTIONAL Sticks 120g (15 pieces)

#### CAUTION

- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In case of contact with eyes, remove and flush with water.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

or for The Americas

<http://www.gcamerica.com>

They can also be obtained from your supplier.

Last revised : 10/2018

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**EU : GC EUROPE N.V.**  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

**GC AMERICA INC.**  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
[www.gcamerica.com](http://www.gcamerica.com)

**GC SOUTH AMERICA**  
Rua Heilodora, 399, Santana - São Paulo, SP, BRASIL  
CEP: 02022-051 - TEL: +55-11-2925-0965  
CNPJ: 08.279.999/0001-61  
RESP. T C: Mayara de Santis Ribeiro - CRO/SP 105.982

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