EN **Prior to use, carefully read the instructions for use.**

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GC Fuji ORTHO™ BAND PASTE PAK

GLASS IONOMER ORTHODONTIC BAND CEMENT

For use only by a dental professional in the recommended indications.

GC Fuji ORTHO BAND PASTE PAK is a glass ionomer orthodontic band cement available in paste-paste delivery.

GC Fuji ORTHO BAND PASTE PAK is available in a cartridge pack and can be dispensed easily and correctly using a special dispenser called Paste Pak Dispenser.

RECOMMENDED INDICATIONS

1. Cementation of orthodontic bands.
2. Cementation of appliances retained with stainless steel clasps (e.g., Herbst or expansion appliances).

CONTRAINDICATIONS

Avoid use on patients with known allergies to glass ionomer cement, methacrylic monomer or methacrylic polymer.

IDENTIFICATION OF PARTS (Paste Pak Cartridge)

1. Cover 2. Inner plunger 3. Cartridge tip (Paste Pak Dispenser)
4. Piston 5. Adjusting guide 6. Lever
7. Sliding block 8. Piston release (Fig. 1)

DIRECTIONS FOR USE

Paste / Paste Ratio (g/g) 2.0 / 1.0
 Mixing Time (sec.) 15"-20"
 Working Time (23°C/73°F) (min., sec.) 3'30"
 Net Setting Time (37°C/99°F) (min., sec.) 3'00"

1. Paste Pak Cartridge Loading

- a. Make sure that the piston is completely retracted into the dispenser. (Fig. 2)
- b. To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them back together.
- c. Load the Paste Pak Dispenser. (Fig. 3)

2. Then turn the cartridge fully to the right position.

- a. To remove the cartridge, retract the piston fully and turn the cartridge back to the marked position.

3. After cartridge loading, push the piston release forward until it stops. (Fig. 4)

- a. Push the piston release forward.
- b. Do not push the upper side.

4. Dispensing

- a. Retract the cartridge cover. (Fig. 5)
- b. Depress the lever to dispense the required amounts of pastes onto the mixing pad.

5. Before dispensing the first material from a new cartridge or when dispensing after a long interval, bleed the pastes in order to prevent the incorporation of air bubbles into the material.

- 1) The adjusting guide can be moved forwards or backwards to change the quantity of material to be dispensed. (Fig. 6)

6. Position of adjusting guide for a smaller quantity:

1. Position of adjusting guide for a smaller quantity.
2. Position of adjusting guide for a larger quantity.

7. While depressing the lever, level and cut off the excess material.

- 1) Level and cut off the excess material by moving the cartridge and dispenser to an upright position on the mixing pad. (Fig. 7)
2. Pressing the lever.

8. When the lever is released after dispensing, residual pastes in the dispenser are retracted into the cartridge. Replace the cartridge cover.

- a. If any surplus paste is retained on the cartridge tips after dispensing, remove with gauze, etc.

9. The amount of residual material in the cartridge can be confirmed by the position of the sliding block on the underside of the dispenser.

- a. When the sliding block has translucent tips, the residual amount can also be confirmed by the position of the inner plunger seen through the cartridge tips.
- b. Do not drop the dispenser with the cartridge.

10. Mixing

After dispensing, incorporate and spread the material out in a thin layer on the mixing pad using a plastic spatula. Mix thoroughly, with tapping strokes for 15-20 seconds. Be careful not to incorporate air bubbles.

11. Band preparation

- a. After adapting the bands, remove any contaminants from the internal surfaces.
- b. For improved retention, roughen the internal surfaces of the bands with a rotary instrument or by sandblasting or microetching.

12. Tooth cleaning

- a. Clean the tooth surfaces to be cemented with pumice and water.
- b. Rinse thoroughly. Blot away excess water with a cotton pellet or sponge. DO NOT DESICCATE. (Enamel surfaces should appear moist (glistening).)

13. Bands

- 1) Using a suitable instrument, place the material through the internal surface of the bands.
- 2) Seat the bands.
- 3) When bands are in the final position, begin to retract the piston and then retract the piston.

14. Appliance

- a. To facilitate removal of the appliance, coat the occlusal surfaces of the teeth with a fluoride gel or petroleum jelly prior to cementation.
- b. Fill the adapted crown with cement and seat. Using a brush or syringe, begin removing excess cement after initial set.

15. Debonding procedure

Remove bonded material from the buccal or lingual side using band removing pliers. Then remove residual cement on the tooth using adhesive removing pliers. And further, remove a thin layer of cement adhering to the tooth using an ultrasonic scaler.

16. Cleaning / Sterilization of Paste Pak Dispenser

1. Clean the dispenser and the adjusting guide with gauze soaked in alcohol.
2. The dispenser may be steam autoclaved at 121-135°C (250-275°F) for 10-30 minutes after disinfection using alcohol. (Please remove the adjusting guide and the cartridge before autoclaving the dispenser). The use of various chemical disinfectants (sterilants) may cause damage to the dispenser and therefore is not recommended.

17. After autoclaving, the piston release of the dispenser may not easily be moved forwards and backwards by fingers. In such a case, slide the piston release forward until it stops, then retract the piston release. Apply a very small quantity of petroleum jelly onto the entire surface of the piston. Do not apply excess petroleum jelly between the piston release and forth a few times to ensure smooth movement.

SHADE
Blue

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C) (39,2-77,0°F).

PACKAGE

Paste Pak Cartridge (13.3 g / 7.2 mL) (2) with mixing pad (No.22)
 Option: Paste Pak Dispenser (1)
 Starter Kit (S.K) only
 Paste Pak Cartridge (13.3 g / 7.2 mL) (2), Paste Pak Dispenser (1), mixing pad (No.22) and plastic spatula (No.23)

CAUTION

1. Avoid excessive contact with oral tissues, removing contact with contact gauze and rinsing with water after initial set.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In rare cases metal may cause sensitivity to some persons. If any such reactions are experienced, discontinue the use of the product and refer to a physician. (nickel-plated aluminum) and refer to a physician.
4. Store Paste Pak Dispenser away from chemicals such as iodine tincture containing iodogen element. Otherwise the dispenser may discolor.
5. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
6. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available.
<http://www.gceurope.com>
 or for the Americas
<http://www.gcamerica.com>

They can also be obtained from your supplier.

CLEANING AND DESINFECTING MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and look for deterioration. Discard device if damaged.

DO NOT IMMERSRE. Thoroughly clean device to prevent drying and accumulation of contaminant. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects: Reporting. If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/orthodontic-devices/contacts_en as well as to our internal vigilance system: vigilance@gc dental
 In this way you will contribute to improve the safety of this product.

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