



A perfect fit for ultimate comfort

GC RELINE
Soft and
Extra Soft

GC

GC RELINE Soft & GC RELINE Extra Soft

Ready-to-use, temporary soft silicone based reline materials.

- Vinyl polysiloxane materials in cartridges offer **easy direct dispensing**
- Surfaces remain **smooth and clean** over time
- Suitable for both chairside and laboratory relining



GC RELINE Soft

- Helps to **compensate resorption** processes
- **Resilient** – acts as a cushion between the denture and oral tissue to buffer masticatory pressure and solve anatomical problems

GC RELINE Extra Soft

- Helps wound healing due to an **extremely soft** consistency
- **Normalises irritated tissue** before you produce a new denture, reline or rebase



Patient Friendly

The use of addition silicon technology offers patients several important benefits:

- Patients can resume social activities within minutes
- No plaque and bacterial deposits – a familiar problem with methyl methacrylates as a result of porosities
- Simple and problem-free cleaning of relined denture




Ease of Handling

- Optimised working time with a snap set means the relining can be finished and polished just 5 minutes after commencing application
- The finished surface is smooth and comfortable. Small adjustments can be made with GC RELINE MODIFIER



Bond strength

GC PRIMER R provides a strong and durable bond between the relining material and denture acrylic.

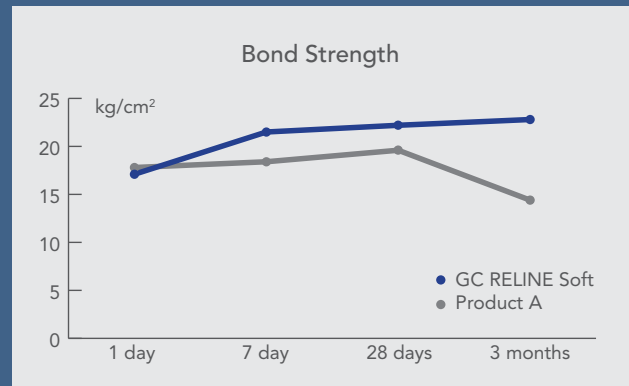

Soft relining

When?

- Presence of **bony prominence** on the maxillary or mandibular arch
- Severe **undercuts** on the ridge
- **Knife edge** mandibular ridge
- Whenever **additional suction** is required
- After the tissue healing phase in **implant** techniques

Why?

- Very thin and resorbed mandibular ridges (common in more mature patients) indicate that the overlying mucosa is very thin and will easily be traumatised by the hard denture base
- Severely resorbed lower ridges may expose the frenulum to the pressure of the denture base – this area will need a soft relining cushion

How?

- Reline the denture with **GC RELINE Soft**

Clinical Case with GC RELINE Soft



Relieve the area to be relined



Apply GC RELINE Primer and gently dry



Apply GC RELINE (Extra) Soft. Working time is 2 mins at 23°C



Distribute over the denture with a spatula



Seat denture in mouth and muscle trim. Retain in situ for 5 mins



Trim excess with an appropriate instrument



Finish with a GC RELINE SILICONE POINT (< 10,000 RPM)



For optimal smoothness, apply mixed GC RELINE MODIFIER A & B

Implantology

When?

- Partial denture (1-3 elements) after implant surgery
- Immediate prosthesis after extraction to facilitate healing before implant insertion
- After bone augmentation procedures
- Relining of complete denture after implant surgery



Why?

- Improve retention of prostheses after surgery
- Reduce pain by uniformly distributing the occlusal load on the denture bearing mucosa
- Aesthetic purposes especially in the anterior region
- Avoid traumatic pressure at the implant site

How?

1. Two-step technique for submerged implants and immediate dentures after extraction.

Firstly, condition the tissue with **GC RELINE Extra Soft** or **GC TISSUE CONDITIONER**. Do this 30-45 minutes after surgery and leave for about 2-3 weeks. Follow with a soft relining for 3-4 weeks with **GC RELINE Soft**.

2. One-step technique around the implant healing screws so the gingiva heals.

Condition the tissue 30-45 minutes after surgery with **GC RELINE Extra Soft** or **GC TISSUE CONDITIONER**. Leave for about 2-3 weeks. In the final stage of treatment, soft reline with **GC RELINE Soft**.

3. In the case of a small posterior edentulous area, avoid giving any temporary removable dentures (if the patient agrees). Ideally do a direct soft reline.



GC RELINE Soft & GC RELINE Extra Soft

Intro Pack

62 g (48 ml) GC RELINE Soft Cartridge

6 Universal Mixing Tips II size L (blue)

13 ml GC RELINE PRIMER R

GC RELINE MODIFIER A & B 10 g (9 ml) each

GC RELINE POINT for trimming

GC RELINE WHEEL for finishing

Accessories: Brush no.7, Plastic spatula, Mixing pad, Sponge, Plastic spatula No.1

Refills

- 62 g (48 ml) GC RELINE Soft Cartridge
- 56 g (48 ml) GC RELINE Extra Soft Cartridge
- 12 g (13 ml) GC RELINE PRIMER R
- GC RELINE MODIFIER, 1-1 pack, A & B 10 g (9 ml) each, Mixing pad (No.15) and sponge
- GC RELINE POINT for trimming, 3 pieces
- GC RELINE WHEEL for finishing, 12 pieces
- Cartridge Dispenser II, 1 pce
- Universal Mixing Tips II Size L (blue), 60 pcs



GC Australasia Dental Pty Ltd
1753 Botany Road, Banksmeadow,
NSW 2019, Australia

T: +61 2 9316 4499
F: +61 2 9316 4196
www.gcasia.info

Copyright © 2013