

everStick® ORTHO

Fibre reinforcement for aesthetic orthodontic retainers

DISTRIBUTED BY

GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku,
Tokyo 174-8585, Japan

GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240,
Interleuvenlaan 33,
B-3001 Leuven, Belgium
TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL
60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1
#06-01 Tampines Plaza
Singapore 529541
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Banksmeadow,
NSW 2019, Australia
TEL: +61 2 9301 8200

GC SOUTH AMERICA
Rua Heliadora, 399,
Santana - São Paulo, SP, Brasil
CEP: 02022-051
TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TÈC: MAYARA DE SANTIS
RIBEIRO - CRO/SP 105.982

RESPONSIBLE MANUFACTURER IN
CANADA
GC AMERICA INC.
3737 W. 127th Street, Alsip, IL 60803
U.S.A.

MADE IN FINLAND



Fibre type: Silanated E-glass fibre impregnated with bis-GMA and PMMA

Form: Unidirectional fibre bundle

Diameter: ~ 0.7-0.8 mm



Keep away
from sunlight



Temperature
limit

Caution: US Federal law restricts this device to sale by or on the order of a dentist.

951330 - NU7007
30000526

everStick® ORTHO



- EN FIBRE REINFORCEMENT FOR AESTHETIC ORTHODONTIC RETAINERS
- DE GLASFASERVERSTÄRKUNG FÜR ÄSTHETISCHE KIEFERORTHOPÄDISCHE RETAINER
- FR FIBRES DE RENFORCEMENT POUR CONTENTION ESTHÉTIQUE ORTHODONTIQUE
- IT RINFORZO IN FIBRA PER L'ORTODONZIA EVERSTICK® ORTHO: PER RETAINER ORTODONTICI ESTETICI
- ES FIBRAS DE REFUERZO PARA RETENEDORES ORTODÓNTICOS ESTÉTICOS
- NL GLASVEZELVERSTERKING VOOR ESTHETISCHE ORTHODONTISCHE RETAINERS
- DA FIBERFORSTÆRKEDE RETAINERS TIL ÆSTETISK ORTODONTI
- SV FIBERFÖRSTÄRKNING FÖR ESTETISKA ORTODONTISKA RETAINERS
- PT REFORÇO DE FIBRA PARA APARELHOS ORTODÓNTICOS ESTÉTICOS
- EL ΕΝΙΣΧΥΣΗ ΜΕ ΙΝΕΣ ΥΑΛΟΝΗΜΑΤΩΝ ΓΙΑ ΑΙΣΘΗΤΙΚΕΣ ΟΡΘΟΔΟΝΤΙΚΕΣ ΑΚΙΝΗΤΟΠΟΙΗΣΕΙΣ
- FI KUITULUJITE ESTEETTISEEN ORTODONTTISEEN RETENTIOON
- ZH 用于正畸治疗后固位阶段的复合树脂纤维
- ID SERAT PENGUAT BAGI RETAINER ORTHODONTIK YANG ESTHETIK

GC


